



HAIR SAMPLING FOR GENOMIC AND ROUTINE PARENTAGE TESTING

Please Read The Following Instructions

Tail hair root follicles are an excellent source of DNA for Genomic and Routine Parentage Testing, however, it is VITAL to observe the following instructions in order to avoid the inconvenience of repeat samples.

- (1) It is essential that at least **40-60** hairs are pulled (**NOT CUT**).
- (2) It is important that the hair is completely **dry** and as **clean** as possible.
- (3) When more than one animal is to be sampled extreme care must be taken to avoid contamination of hair roots from one animal to another.
- (4) Samples should be placed in the provided packet, sealed and sent immediately in the accompanying addressed envelope avoiding prolonged exposure to direct sunlight. (Please ensure the packet is labelled with the animal details etc.)

Thanking you for your co-operation,

Weatherbys Scientific Customer Support team

If you have any questions we are here to support you:

Phone + 353 (0)45 875521 (9:00am to 5:00pm, Monday to Friday, Western European Time Zone UTC+0:00)

e-mail moreinfo@weatherbys.ie

Web weatherbysscientific.com/contact-page/